


Morpheus8: The Future of Skin Rejuvenation & Fat Remodeling

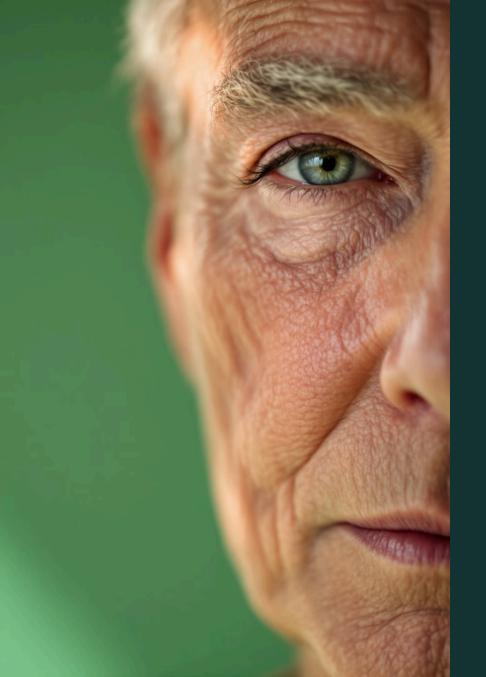

The Problem: Aging Skin

Collagen Loss

Natural decrease in collagen production over time.

Increased Laxity

Skin becomes less firm and elastic.

Jowl Formation

Sagging skin along the jawline creates jowls.


Understanding Morpheus8


- Radiofrequency Energy

 Tiny needles deliver RF energy deep into the skin.
- Natural Healing Response

 Heat triggers the body's regenerative processes.
- Multi-Level Treatment

 Energy deployed in a precise, multi-level pattern.

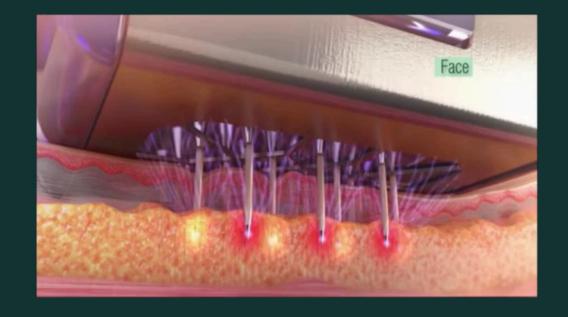

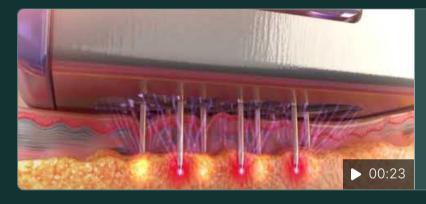



Morpheus8 Radiofrequency Microneedling

Animation of how Morpheus8 by InMode works.

Step-by-Step Process: How Morpheus8 Works

Phase 1: Initial Heating (40-50°C)

For Skin Tightening: RF energy gently heats the dermis, triggering collagen production and improving skin elasticity.

Fat Reduction: The RF energy is delivered deeper (4-7mm) to target subcutaneous fat cells while preserving the skin above.

Phase 2: Thermal Coagulation (50-80°C)

For Skin Tightening: At this stage, collagen fibers contract, making the skin firmer and reducing sagging.

For Fat Reduction: Fat cells begin to coagulate and shrink, leading to gradual fat reduction without damaging surrounding tissues.

Phase 3: Collagen Denaturation (80°C)

For Skin Tightening: Old collagen fibers break down (denature), forcing the body to produce new, stronger collagen, which lifts and firms the skin.

For Fat Reduction: Fat cells undergo apoptosis (cell death), allowing the body to naturally eliminate them through the lymphatic system over time.

Phase 4: Controlled Ablation (100°C)

For Skin Tightening: At higher temperatures, superficial skin layers are gently resurfaced, improving texture, reducing scars, and enhancing glow.

For Fat Reduction: The combination of heat and microneedling ensures that as fat cells shrink, the skin remains firm and tight, preventing post-fat-loss laxity.

Morpheus8 Face



Non-Surgical Facelift & Jawline Sculpting

Advanced skin tightening and lifting technology provides remarkable results for face and neck, effectively reducing jowls and defining the jawline without surgery.



Double Chin & Skin Tightening

Precisely targets stubborn fat deposits while simultaneously preventing skin laxity through enhanced collagen stimulation.


Natural Results

Strategic energy placement ensures optimal sculpting and remodeling, delivering consistent, natural-looking results.





No Surgery, No Downtime

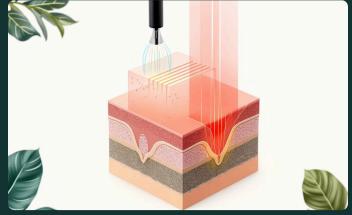


Advanced Fat & Skin Remodeling with less downtime



Maximum Versatility & safe for all skin types with multiple depth and controlled heat

Aesthetic Benefits

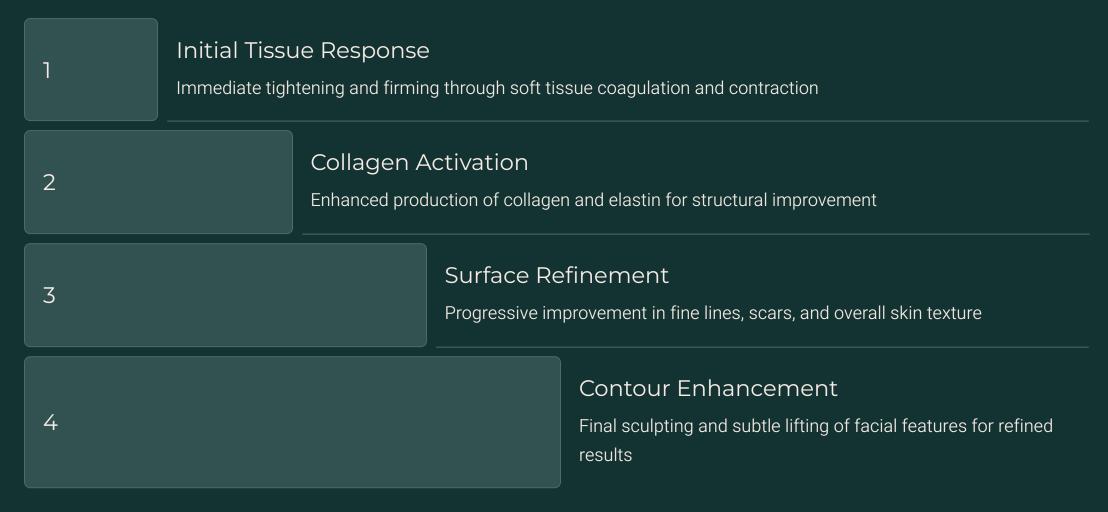

These benefits work together progressively to create comprehensive skin rejuvenation, with results that continue to improve over time.

Wound Healing Cascade

Inflammation

Increased enzyme and cytokine activity.

Proliferation

Enhanced immune system response.

3 Remodeling

Removal of damaged tissue, deposition of new cells.



Morpheus8 Burst Deep Technology

 $\begin{vmatrix} 1 & \rangle \end{vmatrix}$ 2 $\begin{vmatrix} 3 & 2 & 2 \end{vmatrix}$



Deepest RF Energy

Provides the most targeted deep RF energy deposition available



Multi-Level Contraction

Delivers deep, full-thickness soft tissue and dermal contraction



Intelligent Technology

Uses programmable SCALE and BURST technology for optimal results

This advanced technology works synergistically to deliver unprecedented depth and precision in skin remodeling treatments.

The Solution: Morpheus8

1

Non-Surgical Facelift

Alternative to traditional surgical procedures.

2

Deep Tissue Remodeling

Works beyond the surface for comprehensive results.

3

Fat Remodeling & Skin Tightening

Only device that can do Both





Ideal Candidates



Fine Lines

Reduces appearance of fine lines and wrinkles.



Sagging Skin

Tightens and lifts loose skin.



Jowls

Improves definition of jawline.



Acne Scars

Smooths and improves texture of scarred skin.

Debunking Myths

Fat Melting Myth

RF microneedling doesn't melt fat. It's about precise energy placement.

Provider Expertise


Results depend on the provider's knowledge of anatomy and technique.

Lifting, Not Volume Loss

Proper treatment focuses on lifting, not fat reduction.



The Treatment Process

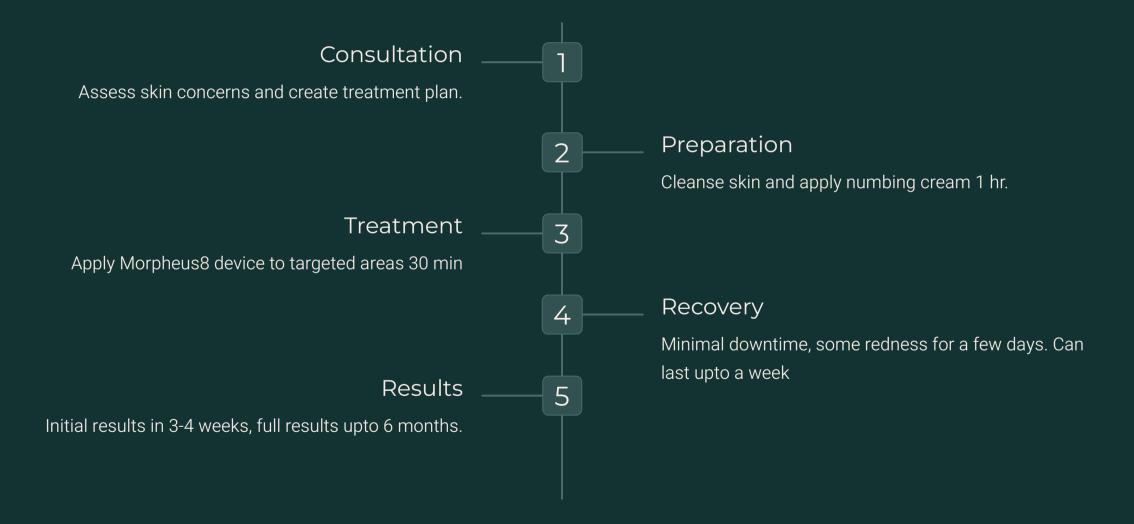

Morpheus8 Treatment Packages

Choose from our comprehensive treatment packages with exclusive membership savings

Face & Neck Treatments

Single Session: \$1,200

Tier 1 Members: \$960

Tier 2 Members: \$840

Tier 3 Members: \$720

3-Session Package: \$3,450

Tier 1 Members: \$2,760

• Tier 2 Members: \$2,415

• Tier 3 Members: \$2,070

Double Chin Treatments

Single Session: \$600

• Tier 1 Members: \$480

• Tier 2 Members: \$420

• Tier 3 Members: \$360

3-Session Package: \$1,650

Tier 1 Members: \$1,320

Tier 2 Members: \$1,155

• Tier 3 Members: \$990

Stomach Treatments

Single Session: \$1,200

• Tier 1 Members: \$960

Tier 2 Members: \$840

• Tier 3 Members: \$720

3-Session Package: \$3,450

Tier 1 Members: \$2,760

Tier 2 Members: \$2,415

• Tier 3 Members: \$2,070



Frequently Asked Questions

Is it painful?

Minimal discomfort with numbing cream provided.

How many sessions?

3 sessions recommended, visible change after 1.

Downtime?

Slight redness for a few days, no major downtime.

Results duration?

Results can last upto 2 years with proper skincare. Yearly 1 touch ups recommended

A Personal Thank You from Me & My Team

To my incredible clients,

Thank you for trusting **me and my team** with your skin, your confidence, and your journey to feeling your best. Every time you walk through our doors, you're not just a client—you're part of something I've built with passion, integrity, and a deep love for what I do.

I promise to always be honest, to guide you with care, and to deliver results that you'll love—not just today, but for years to come. My team and I are here for you, every step of the way.

With all my gratitude,

Sapana Dhakal

Founder, **ILML Aesthetics**